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O2 Sensor: Calibration needed

Circuit Check: Calibration needed

EXH Flow Sensor: Calibration needed

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1. กดปุ่ม Calibrations

The screenshot shows the 'New Patient Startup' screen of the eVent Medical 3e software. The interface is dark-themed with blue accents. At the top left, 'eVent Medical' is displayed. At the top right, 'eVolution 3e' is shown. Below the top bar, there's a section for 'Patient Type: Adult' and a field for 'Patient ID: Room/Bed ID:'. The main area is divided into two columns. The left column, titled 'Patient Option', contains buttons for 'New Patient' (with sub-options 'Adult', 'Pediatric', 'Neonate') and 'Current Patient'. The right column, titled 'New Patient Startup', contains fields for 'Ideal Body Weight (IBW)' in kg, 'Patient Height' in cm, 'Gender' (set to 'Male'), and 'Humidification Type' (set to 'HME'). A yellow arrow points from the 'Calibrations' button in the right column to the '1. กดปุ่ม Calibrations' text box. At the bottom left, there's a status section with 'O2 Sensor: Calibration needed', 'Circuit Check: Calibration needed', 'EXH Flow Sensor: Calibration needed', and 'System Compliance: — ml/cmH2O'. At the bottom right, there's an 'ACCEPT' button and 'Software Versions - 2.0.0 - 2.1.1 - 6.5.00'.

eVent Medical

eVolution[®] 3e

Patient Type: Adult

ID Patient ID:
Room/Bed ID:

Patient Option

New Patient

Adult

Pediatric

Neonate

Current Patient

New Patient Startup

Ideal Body Weight (IBW)

IBW Patient Height

kg cm

Gender

Male

Humidification Type

HME

Calibrations

ACCEPT

O2 Sensor: Calibration needed
Circuit Check: Calibration needed
EXH Flow Sensor: Calibration needed
System Compliance: — ml/cmH2O

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